

# Denracle Library Installation for EXOCAD

Follow the below steps to install Denracle Library in your exocad.

## STEP 1

Download Exocad Denracle library, “ExoCAD\_Denracle V4.0.rar”, from Denracle official website (denracle.com) or contact our sales rep. for the file.

## STEP 2

Unzip the file, “ExoCAD\_Denracle V4.0.rar”.

## STEP 3

Enter the folder and see the Denracle database for all available systems as below.

## STEP 4

Copy and paste all the system files in the index for the Exocad libraries. Path: DentalCADApp>library>implant

## STEP 5

The Denracle Library is installed.

